
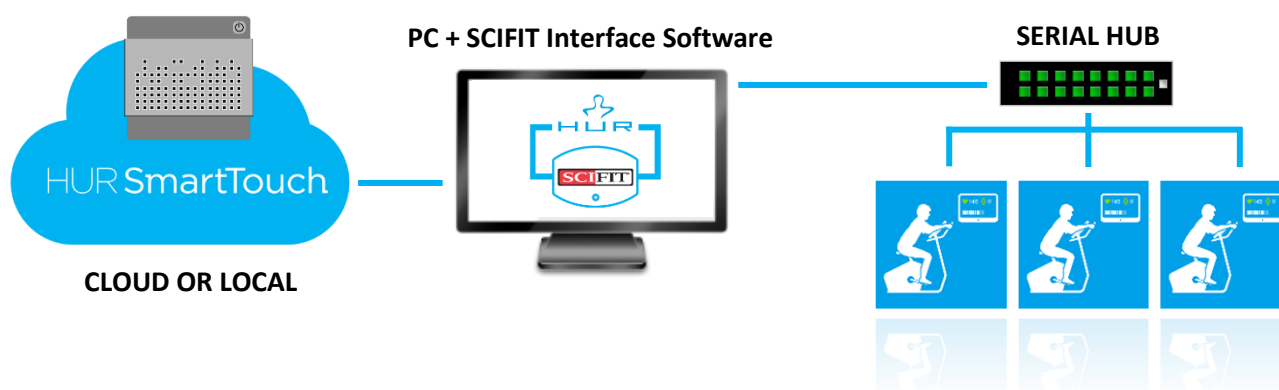



## REQUIREMENTS

<p>SCIFIT Intelli FIT touchscreen console with at least version <b>3.01</b> or SCIFIT console with version <b>RT2.80B</b></p>	
<p>Consoles that support HUR SmartTouch <u>have RFID reader sticker</u></p>	
<p>Serial HUB (Up to 8 Scifit machines per HUB)  Or Digitus Serial-USB port adapter</p>	
<p>SCIFIT Cables (RJ-45 to serial) - 1 cable per Scifit machine</p>	
<p>Windows PC ( with enough USB ports available) &amp; SCIFIT Interface Software</p>	
<p>SCIFIT equipment and activities added to the HUR SmartTouch software</p>	



## INSTALLATION


1. Connect SCIFIT machines to the serial hub with SCIFIT Cables (RJ-45 to serial)

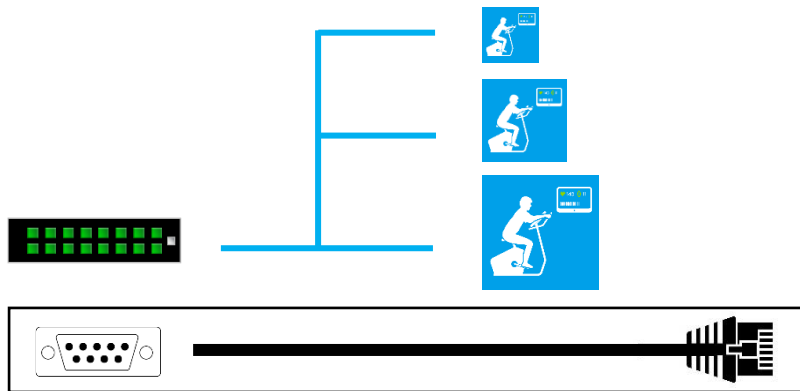

2. Connect serial hub to the Windows computer with USB cable.



3. Install drivers for serial hub. (Included on CD-rom).

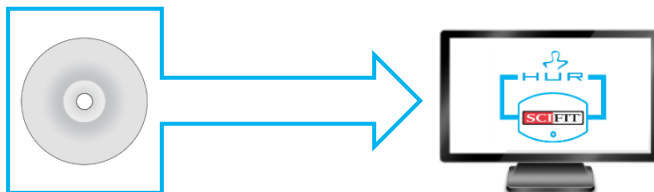


4. Install the SCIFIT Interface Software.


(<http://software.hur.fi/software/DL/INTERFACES/HURSCIFITInterface.exe> )



5. Start service control program **Scifit Interface.exe** from the shortcut on the desktop. Remember to run it as admin. Right Click on the file with the mouse and Select "Run as administrator"

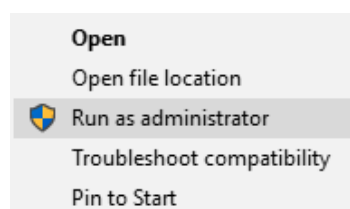

6. Set the correct HUR SmartTouch server address, username and password. Save the new settings. (Restart software or) and close (starts the service).

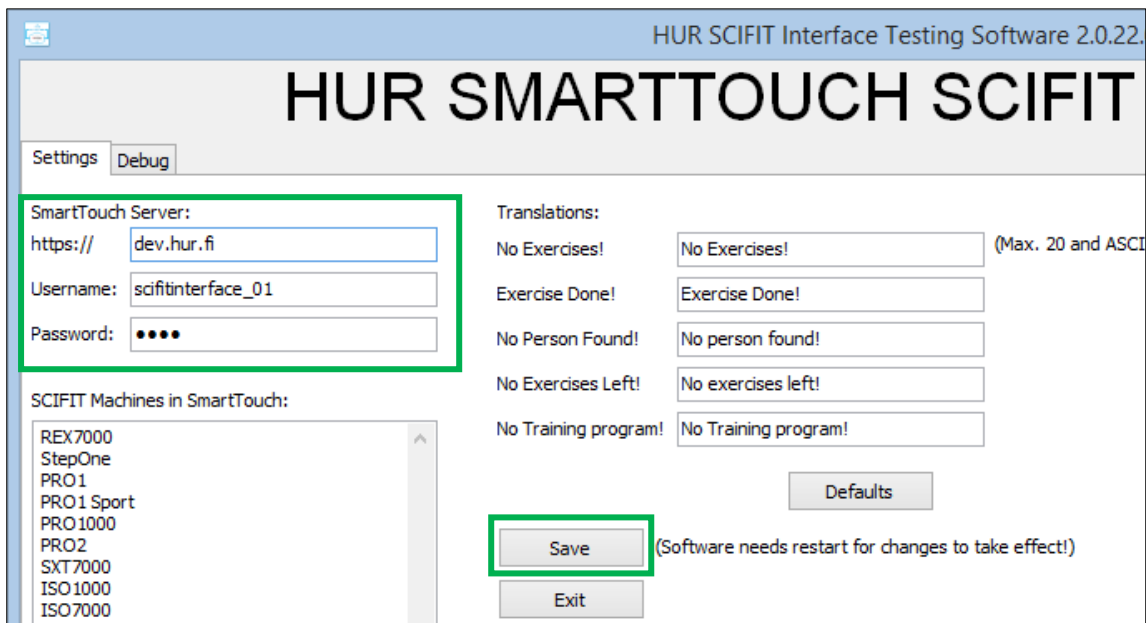

HUR SCIFIT Interface Testing Software 2.0.22.

## HUR SMARTTOUCH SCIFIT

Settings Debug

SmartTouch Server:

https://

Username:

Password:

Translations:

No Exercises!  (Max. 20 and ASCII)

Exercise Done!

No Person Found!

No Exercises Left!

No Training program!

Defaults


Save (Software needs restart for changes to take effect!)

Exit

SCIFIT Machines in SmartTouch:

- REX7000
- StepOne
- PRO1
- PRO1 Sport
- PRO1000
- PRO2
- SXT7000
- ISO1000
- ISO7000

7. In order to have the HUR SCIFIT Interface service running without someone logging in to the computer, go to HUR SCIFIT Interface service settings and set account settings. Select 'This account' and clear password. Use 'Network Service' as the account.

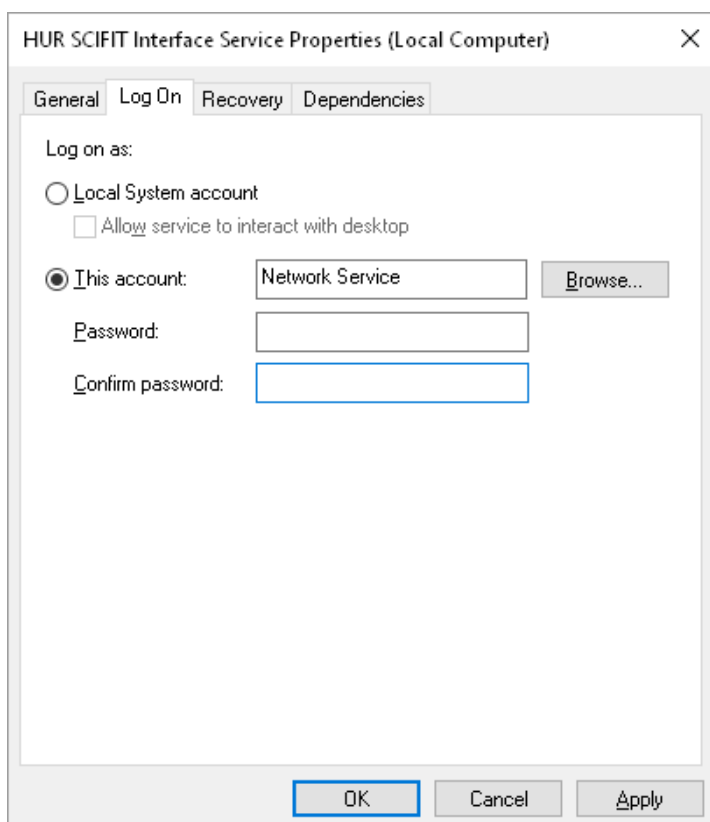

HUR SCIFIT Interface Service Properties (Local Computer)

General Log On Recovery Dependencies

Log on as:

☐ Local System account

☐ Allow service to interact with desktop

☒ This account:  Browse...

Password:


Confirm password:

OK Cancel Apply

8. Restart the computer.



9. Make sure that SCIFIT machine has its power supply connected.

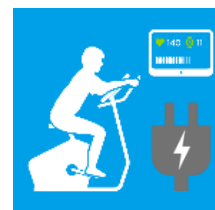

10. Test all devices by presenting a RFID card/bracelet to RFID sticker on SCIFIT consoles.

## TROUBLESHOOTING:

Problem	Checklist
<b>None of the SCIFIT machines react to the RFID tag.</b>	<ul style="list-style-type: none"> <li>• Is the computer running where the service is running?</li> <li>• Is the service running on this computer?</li> <li>• Do all of the SCIFIT machines have power adapter in use?</li> <li>• Check drivers for Hub/Adapter.</li> </ul>
<b>Individual machine doesn't react to RFID tag.</b>	<ul style="list-style-type: none"> <li>• Is the machine properly connected from the machine's <u>COM port</u> to the computer?</li> </ul>
<b>RFID reading range is poor or the position doesn't match with the sticker.</b>	<ul style="list-style-type: none"> <li>• The reader inside the console might not be perfectly matched with the sticker.</li> <li>• Reposition the sticker if needed.</li> <li>• Consult SCIFIT support regarding the console's internal RFID reader.</li> <li>• Try with RFID card (if you have one) instead of RFID bracelet</li> </ul>